




Installation and Setup Guide




20090v1.01

Versa Support: versasupport@prc-salttillo.com

Contents

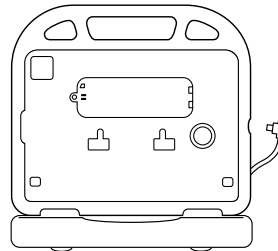


Front
View

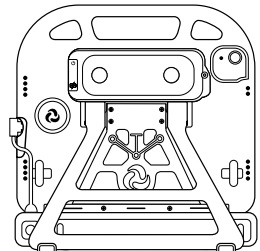

Back View*


*Shown with optional
VersaSpeaker accessory

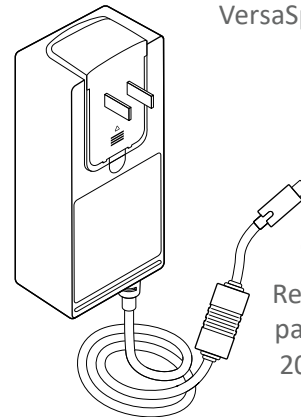


Charger

Replacement
part number
20540v1.00

Installation

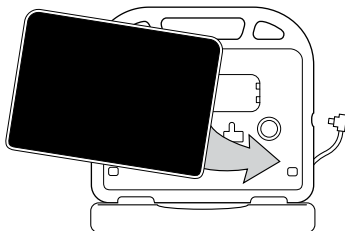

Insert iPad®
(iPad sold separately)

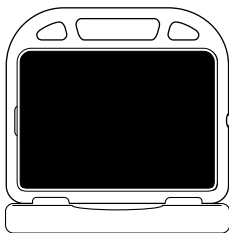

Plug in Cable

1. The wrap fit is snug for the protection of the iPad. Set bottom of iPad at angle and insert right corner. Then press the left bottom corner into place. Carefully work the upper right corner of iPad into the case followed by the upper left.
2. Insert USB-C cable from VersaEye into the USB-C port of the iPad.
3. Start your iPad.

1

Setup


Connect to Wi-Fi



Open the App Store



Search IRISBOND HIRU

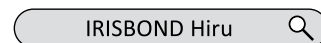

Select GET, but don't open yet




GET

2

Charging

Insert the included charger. This will charge the iPad and the VersaEye batteries.

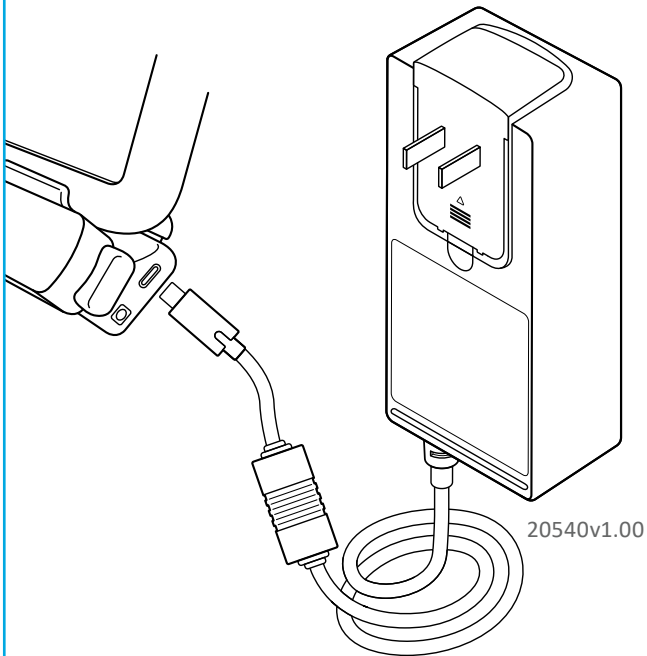

3

Power Button and LEDs

Turn VersaEye ON/OFF: Hold  for 3 seconds


-  1 Green LED: Powering ON
-  1 Red LED: Powering OFF

 Battery Status: Tap 

-  All 3 LEDs lit if 60% +
-  2 LEDs lit if below 60%
-  Only 1 LED lit if below 30%


Charging

-  1 LED Flashing: Charging
-  1 LED Solid: Fully charged

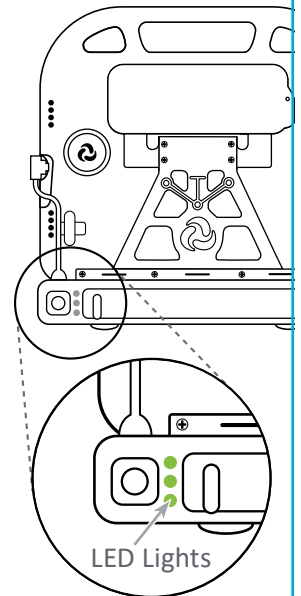

4

Hiru Update Settings


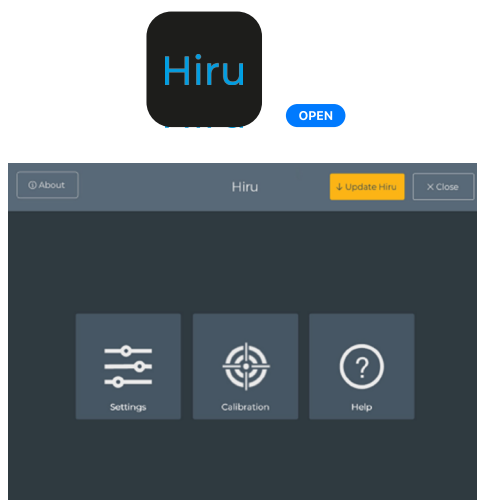


If yellow Update Hiru button appears:


1. Set Auto-Lock to Never in Settings > Display & Brightness during updates.
2. Return to Hiru app and proceed with update.
3. Restart iPad and VersaEye module.

5

Positioning

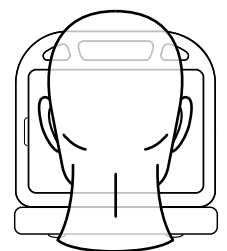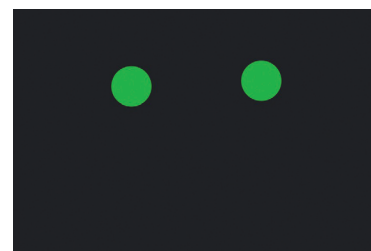

Align your eyes with the upper 1/3 of the iPad screen.

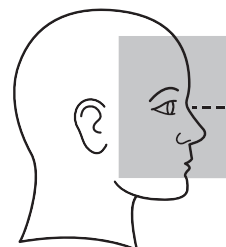


Sit approximately 22" from the device.



6


Calibration

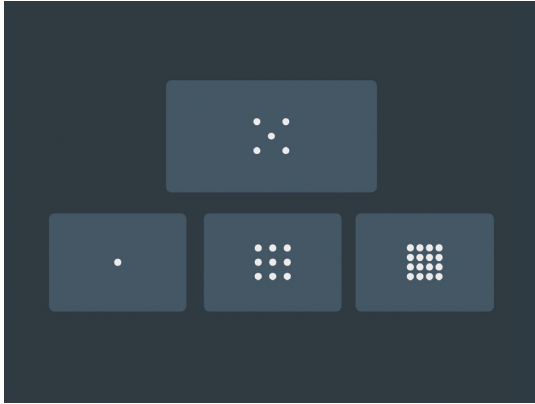

The number of Calibration Points determines the number of stimulus points you will look at during calibration. The more points you follow accurately, the more accurate control the user will gain over more of the screen.

7

Adjust Smoothing

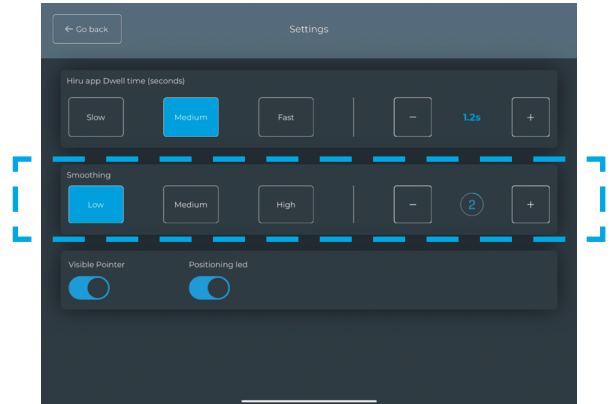

- Smoothing changes the sensitivity of the cursor.
- Lower values will be more sensitive to eye and head movements.
- Higher values will be less sensitive to eye and head movements.

8

Use a Switch to Select

1. Plug in your switch or connect it via Bluetooth®.
2. Move the cursor to the target using your eyes.
3. Activate the switch to Tap, Double Tap, or Hold and Drag.

VersaEye is compatible with a 3.5 mm wired or Bluetooth wireless switch for making selections.

Use Dwell Control to Select

1. Go to: Settings > Accessibility > Touch > AssistiveTouch.
2. Turn on Dwell Control.
3. Movement Tolerance: Adjust the distance the cursor can move while dwelling on an item.
4. Adjust Dwell Time.



9

For More Versa and AAC Information:



AAC Apps
aacapps.com



AeroSwitch®
store.prc-salttillo.com/
product/aeroswitch



Mounts
store.prc-salttillo.com/
category/mounts-brackets



VersaSpeaker
store.prc-salttillo.com/
product/versaspeaker

For warranty & additional information, visit:
prc-salttillo.com/versa

VersaEye is a trademark of PRC-Salttillo. ©2026 PRC-Salttillo. All other trademarks and copyrights are the property of their respective owners. All rights reserved.

10